
eMyScale

minilandbaby.com

miniland
BABY

Español	9	Deutsch	29
English	14	Italiano	35
Português	19	Polski	40
Français	24	Русский	45

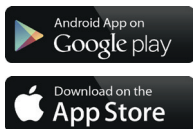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TABLE OF CONTENTS

1. INTRODUCTION
2. CONTENT
3. SAFETY INSTRUCTIONS
4. PRODUCT CHARACTERISTICS
5. INSTRUCTIONS FOR USE
6. MAINTENANCE
7. TECHNICAL SPECIFICATIONS
8. INFORMATION ON THE DISPOSAL OF THE BATTERIES AND THE PRODUCT

■ 1. INTRODUCTION

We would like to congratulate you on your purchase of eMyScale, scales for babies and children of up to 20 kg in weight, connected to your smartphone or tablet via Bluetooth using the exclusive eMyBaby application to register your baby's weight. Read the instructions manual carefully prior to using the apparatus..

Keep this manual for future use as it contains important information.

The characteristics described in this user guide are subject to change with no prior warning.

■ 2. CONTENT

1 set of scales

3 AAA batteries

1 instructions manual and guarantee

If any of the aforementioned contents are missing, contact your distributor.

■ 3. SAFETY INSTRUCTIONS

Read the safety instructions carefully prior to using this apparatus and keep it for future reference.

1. You should not strike the scales with sharp or hard objects and avoid strong vibrations.
2. Avoid heavy objects which surpass the maximum capacity of the scales (20kg/44lb/1.48oz).
3. Never wet the scales. The scales should only be cleaned with a smooth cloth and natural cleaning products.
4. Protect the scales well. If the surface is damaged the readings may be inaccurate.
5. Replace the batteries when the 'LO' indicator appears on the screen.
6. The batteries should be inserted with the poles in the correct position.
7. Remove the batteries before storing the scales for long periods of time.
8. Do not attempt to open or disassemble any part of the device except to replace the batteries.
9. Use the device exclusively for the purpose it is intended for.
10. Do not attempt to repair this product on your own, as opening and handling it may damage the device or expose it to hazards.

N.B: We should warn you that alterations and modifications to the equipment which are not conducted by the MINILAND technical department are not covered under the product guarantee.

■ 4. PRODUCT CHARACTERISTICS

4.1. DESCRIPTION OF THE SCALES (Figure 1)

- a. The housing
- b. Weight unit button (kg/lb/oz)
- c. "HOLD" button

- d. LCD screen
- e. Tare button
- f. ON/OFF button
- g. Battery compartment
- h. Brackets

4.2. DISPLAY (Figure 2)

1. "HOLD" function on/off icon
2. Battery icon
3. Minus sign
4. Tare icon
5. Digits for the weight measurement
6. Measurement unit icons

■ 5. INSTRUCTIONS FOR USE

5.1 PREPARATION

5.1.1. CHANGING THE BATTERIES

Change the batteries whenever the image on the screen is unclear and the word "LO" appears on the display.

1. Remove the cover to the battery compartment (g) located in the scales.
2. Replace the AAA type batteries. Check the batteries are correctly in place in accordance with the diagram in Figure 3.
3. Close the cover to the compartment correctly.

5.1.2. OPERATING ENVIRONMENT

Be sure to place the scales on a firm horizontal surface in order to avoid damage to the equipment and to ensure the accuracy of the readings.

5.2 USING THE SCALES

1. Press the "ON/OFF" button (f) to turn the scales on. All the display (d) items will appear for a few seconds.
2. Press the "Weight Unit" button (b) when "0.00" appears on the screen to set the weight to "kg", "lb" or "oz". The option you have selected will appear on the display (6).
3. The scales are now ready for use.
4. The Tare function enables you to remove the weight of accessories such as towels to prevent registering a false reading when weighing your baby in this manner. You should first weigh these objects and then press the Tare button (e). The word "TARE" will appear on the display and only what is placed on the scales from then on will be weighed.
5. Place the baby on the scales when "0.00" (kg/lb/oz) appears on the screen.
6. The definitive weight will be registered on the display when the word "HOLD" appears. The scales will turn off automatically after one minute.
7. If the baby moves during the process you may use the "HOLD" function. Turn on the scales (conducting the tare process where necessary) and place the baby in position. You will notice the reading on the display varies slightly when the baby moves. Press the "HOLD" button (c) and the word "HOLD" will start to flash on the display. As long as the word "HOLD" is not flashing the scales will read your baby's weight correctly. The scales will turn off automatically after one minute.

ENGLISH

Note: If you place an object on the scales before turning them on, the reading will be “0.00” when they are turned on (due to the fact the scales conduct an automatic tare when they are turned on). Remove the object and press the “Tare” button (e) to give the empty scales a reading of “0.00”.

5.3. eMyBaby

5.3.1. REGISTRATION

1. First of all register on the eMyBaby website. As such: open your browser, access **emybaby.com** and click on the CREATE ACCOUNT button.
2. complete the fields with your personal data, accept the terms and conditions of use and click on the ACTIVATE ACCOUNT button.
3. You will receive a message of confirmation via the e-mail address you provided when registering. Clicking on the link containing your e-mail message will return you to the eMyBaby site, which you will be able to access by providing your username and password after clicking on the ACCESS button.

5.3.2. eMyScale Module from the eMyBaby Website

1. Once you have entered your username and password and accessed eMyBaby, select EMYSSCALE in the OFFICE option.
 2. Select the baby you wish to weigh (registered beforehand on the MY BABY module).
 3. Enter the weight data manually.
- You may consult all the registers and view the progress of your baby’s weight in graph form, comparing them to WHO standards if you wish.

Note: Check the Bluetooth on your mobile device is turned on before proceeding to register the readings via eMyBaby.

5.3.3. eMyBaby Application

Miniland has developed the eMyBaby application, available for iOS and Android, and easy to download free of charge.

In order to download the eMyBaby application you should first access the respective application store or o download it directly by scanning the corresponding QR code from those illustrated below:

Follow the instructions provided at the start of this manual to install the application.

1. Once the application has been installed, open it. Go to LOGIN and enter your username and password to gain access, if you are already registered, or register if you have not done so before. Remember you will receive a message of confirmation via the e-mail address you provided when registering.
2. Select the EMYSSCALE option on the main menu.
3. Select the baby you wish to weigh (registered beforehand on the MY BABY module).
4. Register the weight provided by the scales automatically or enter the weight data manually.

You may consult all the registers and view the progress of your baby's weight in graph form, comparing them to WHO standards if you wish.

5.4. IDENTIFYING AND SOLVING PROBLEMS

PROBLEM	POSSIBLE CAUSES	SOLUTION
Your mobile device is unable to find eMyScale on taking a reading.	Your scales are not turned on.	Turn them on by pressing the "ON/OFF" button (f)
	Your mobile device is not compatible with Bluetooth BLE 4.0.	Check compatibility with eMyScale in section 7, TECHNICAL SPECIFICATIONS
	Your mobile Bluetooth is not turned on.	Check the Bluetooth on your device is turned on.
	The batteries in your scales are flat.	Replace the batteries as illustrated in item 5.1.1.
	eMyScale is out of the Bluetooth range of your mobile device	Place your mobile device closer to the scales
The mobile device does not show the weight when weighing	You have selected the manual register option on eMyBaby	Select the scales register option
The weight reading displayed on your mobile device is incorrect	This may be due to interference during the transmission of the data	Repeat the operation
	The battery level on your scales is too low	Replace the batteries as illustrated in item 5.1.1.
The scales will not turn on	The batteries in your scales are flat.	Replace the batteries as illustrated in item 5.1.1.
	The batteries have not been put in place correctly	Insert the batteries in accordance with the instructions in item 5.1.1.

■ 6. MAINTENANCE

6.1. CLEANING

- Clean the surface of the scales with a smooth cloth and natural cleaning products (when necessary).

Do not use solvents or abrasive detergents.

- Never wet the scales.

■ 7. TECHNICAL SPECIFICATIONS

- Units of weight: kg / lb / oz

- Maximum capacity: 20 kg / 44 lb / 705.48 oz

ENGLISH

- Intervals: 0.01kg / 0.02lb / 0.1oz
- Accuracy: 0.6% (of the weight of the object) $\pm 1d$, whereby $d = 10g$

1 kg	+ 16g / - 4g
2 kg	+ 22g / + 2g
3 kg	+ 28g / + 8g
4 kg	+ 34g / + 14g
5 kg	+ 40g / + 20g
6 kg	+ 46g / + 26g
7 kg	+ 52g / + 32g
8 kg	+ 58g / + 38g
9 kg	+ 64g / + 44g
10 kg	+ 70g / + 50g
15 kg	+ 100g / + 80g
20 kg	+ 130g / + 110g

- Overload indicator: "EEEE"
- Low battery indicator: "LO"
- 1-minute automatic off function
- Bluetooth BLE 4.0
- Compatibility:
 - Android:
Version: 4.3 and later
Devices compatible with BLE 4.0
 - iOS:
Version: 7.0 and later
Devices: iPhone 4S, 5, 5S, 5C, 6 and 6 Plus; iPad Air, iPad Mini and iPad 3rd and 4th generation.
- Batteries: 3 x AAA 1.5V

8. INFORMATION ON THE DISPOSAL OF THE BATTERIES AND THE PRODUCT

- Remember to remove the batteries when disposing of this product and dispose of the same in a responsible manner.
- Do not mix the products with the crossed-out bin symbol with the rest of your domestic waste
- Once the service life of these products has ended they should be disposed of at a collection and treatment plant assigned by the local authorities. You may also contact the establishment at which you purchased the product.

Hereby, Miniland S.A. declares that this "89158 emyscale" is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

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miniland BABY DECLARATION OF CONFORMITY

Miniland S.A. P.Ind. La Marjal I C/ La Patronal 8-10. 03430 ONIL (Alicante) SPAIN
Technical support: +34 966 557 775 · www.minilandbaby.com · miniland@miniland.es

We declare under our own responsibility that the product:

eMyScale · Miniland Baby · 89158 · Miniland, S.A.

to which this declaration refers conforms with the relevant standards or other standardising documents:

- **Directive 1999/05/EC of the European Parliament and of the Council of 9th March Radio Equipment and Telecommunications Terminal Equipment**
 - EN 301 489-1 V1.9.2 (2011-09) Electromagnetic compatibility and Radio spectrum Matters (ERM); Electro-Magnetic Compatibility (EMC) standard for radio equipment and services; Part 1: Common technical requirements
 - EN 301 489-17 V2.2.1 (2012-09) Electromagnetic compatibility and Radio spectrum Matters (ERM); Electro-Magnetic Compatibility (EMC) standard for radio equipment; Part 17: Specific conditions for Broadband Data Transmission Systems
 - EN 300 328 V1.8.1 (2012-06) Electromagnetic compatibility and Radio spectrum Matters (ERM); Wideband transmission systems; Data transmission equipment operating in the 2,4 GHz ISM band and using wide band modulation techniques; Harmonized EN covering the essential requirements of article 3.2 of the R&TTE Directive
 - EN 62479:2010 Assessment of the compliance of low power electronic and electrical equipment with the basic restrictions related to human exposure to electromagnetic fields (10 MHz to 300 GHz)
- **Electromagnetic Compatibility (EMC) Directive 2004/108/EC.**
 - EN 61000-6-1:2007 Electromagnetic compatibility (EMC) — Part 6-1: Generic standards — Immunity for residential, commercial and light-industrial environments
 - EN 61000-6-3:2007+A1:2011+AC:2012 Electromagnetic compatibility (EMC) — Part 6-3: Generic standards — Emission standard for residential, commercial and light-industrial environments
 - EN 55014-1:2006+A1:2009+A2:2011 Electromagnetic compatibility — Requirements for house hold appliances, electric tools and similar apparatus — Part 1: Emission.
 - EN 55014-2:1997+A1:2001+A2:2008 Electromagnetic compatibility — Requirements for house hold appliances, electric tools and similar apparatus —Part 2: Immunity — Product family standard.
- **Restriction of Hazardous Substances (RoHS) Directive 2011/65/EU.**
- **REACH (Registration, Evaluation, Authorization and Restriction of Chemical substances) Regulation (EC) No 1907/2006.**
- **Waste Electrical and Electronic Equipment (WEEE) Directive 2012/19/EU.**

Miniland, S.A.
Chief Executive Officer
José Verdú Francés
Onil, Alicante, Spain,
29th December 2015

miniland
EDUCATIONAL · BABY
minilandgroup.com

Miniland S.A. P.Ind. La Marjal I C/ La Patronal 8-10
03430 ONIL (Alicante) SPAIN

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Made in China. Keep this information.

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© Miniland, S.A. 2015